

# Easy working.

## Instructions... step-by-step

Cut away excess impression material... including any excess height in the palette area.



Put the BAFIX® model-tray onto the impression, checking the correct position of the die(s).



Make referenced marks between the impression tray and the BAFIX® model-tray.



Fill the impression and the BAFIX® model-tray at the same time with the same dental stone, ensuring the levels of both the BAFIX® model-tray and the impression are flush.



Place the BAFIX® model-tray over the dental impression, lining-up the reference marks made previously.



After the stone has hardened, tap around the BAFIX® model-tray to remove the model and separate it from the impression.



With the help of a tool, separate the locking clips thus releasing the model.



Gently tap both the sides and the base of the BAFIX® model-tray.



After removing any remaining stone from around the edges, proceed to cut and adjust the model using your normal technique.