



BAFX1[®]

The best **tray** for your work



Mod. Reg. C.E.
000157177-0001



Development and Technology
of Dental Systems

Desarrollo y Tecnología
de Sistemas Dentales

Julián Camarillo, 7 - 3^º C - 28037 MADRID - Teléfono: +34 913 049 305 - Fax: +34 913 043 924
e-mail: dtds@dtdsystems.com - web: www.dtdsystems.com


Easy working.

Instructions... step-by-step

Cut away any excess impression material... including any excess height in the palette area.



Put the BAFIX model tray onto the impression, checking the correct position of the die(s).

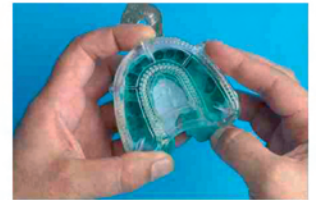

Make referenced marks between the impression tray and the BAFIX model tray.



Fill the impression and the BAFIX model tray at the same time with the same dental stone, ensuring the levels of both the BAFIX model tray and the impression are flush.



Place the BAFIX model tray over the dental impression, lining-up the reference marks made previously.



After the stone has hardened, tap around the BAFIX model tray to remove the model and separate it from the impression.



With the help of a tool, separate the locking clips thus releasing the model.



Gently tap both the sides and the base of the BAFIX model tray.



After removing any remaining stone from around the edges, proceed to cut and adjust the model using your normal technique.